



Solana[®] Strep Complete ASSAY



Enhance your strep testing with Group A and pyogenic Group C/G Strep*

Pyogenic Groups C/G β -hemolytic Streptococcus is another significant cause of acute Streptococcal pharyngitis. However, it will be undetected when using a rapid antigen test that only detects Group A Strep. Solana with Solana Strep Complete detects and differentiates *Streptococcus pyogenes* and *Streptococcus dysgalactiae* in just 30 minutes. The result of the Solana Strep Complete is final without the need for culture back up and can also be used to confirm negative rapid strep A test results.

Prompt initiation of patient management and antimicrobial therapy

- Results in just 30 minutes
- High volume throughput capabilities
Batch up to 6 tests per run

No Culture Back Up

- Gold standard sensitivity
- Confidence in your prescription decision

Ease and Affordability

- Reduced hands-on time vs. culture
- Simple procedure
- Competitive reagent pricing
- Flexible instrument placement options


The Accurate. Sustainable.
Molecular **Strep** Solution.

*Strep dysgalactiae


Solana Strep Complete Assay run with the Solana
bench top instrument a mere 9.4" x 9.4" x 5.9"



Procedure

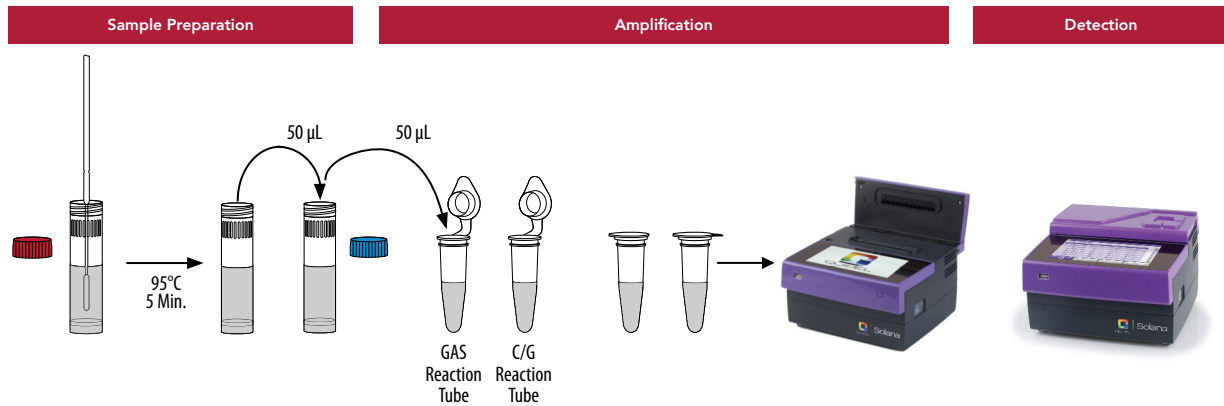

Time to 6 results

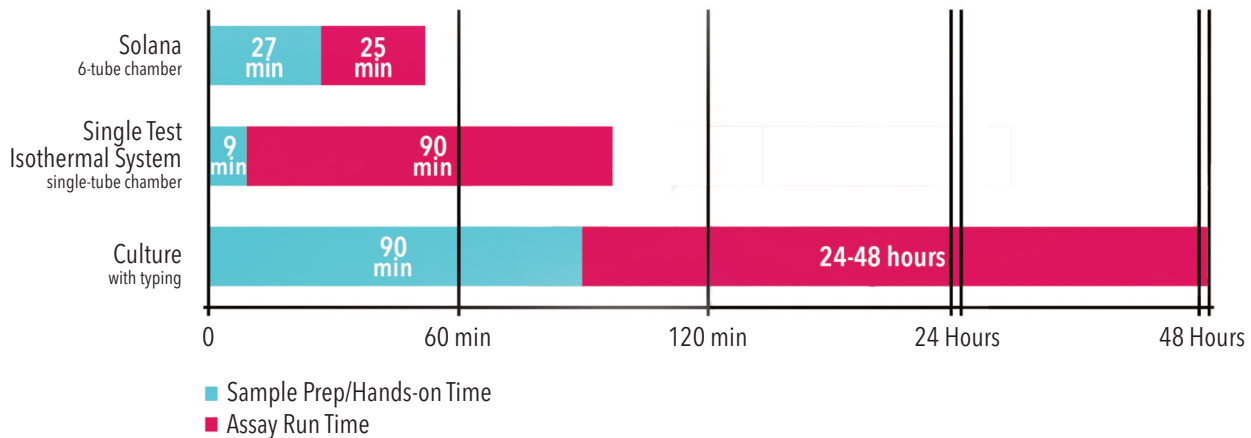

Clinical performance

Performance Results of Solana Strep Complete Assay for *Streptococcus pyogenes*

Combined Culture and NAAT						
Solana Strep Complete		POS	NEG	Total	Sensitivity	98.8%
	POS	475	25	500	Specificity	98.9%
	NEG	6	2180	2186		
	Total	481	2205	2686		

*Refer to Package Insert for additional performance claims.

Performance Results of Solana Strep Complete Assay for *Streptococcus dysgalactiae*

Combined Culture and NAAT						
Solana Strep Complete		POS	NEG	Total	Sensitivity	100%
	POS	78	14	92	Specificity	99.5%
	NEG	0	2594	2594		
	Total	78	2608	2686		

*Refer to Package Insert for additional performance claims.



QUIDEL

Solana Strep Complete Assay – 48 Test Kit: Cat. #23043109
 Quidel Molecular Strep A+G Control Set – 20-Reaction Kit: Cat. #23043150
 Solana Strep Complete Startup Kit: Cat. #23043110

SSM305201EN00 (02/17)

© 2015 Thermo Fisher Scientific Inc. All rights reserved.
 Trademarks used are owned as indicated at www.fishersci.com/trademarks.

In the United States:

For customer service, call 1-800-640-0640
 To fax an order, use 1-800-290-0290
 To order online: www.fisherhealthcare.com



A Thermo Fisher Scientific Brand